

SAMPLE COLLECTION USER FLOW



Do not eat, drink, smoke 30 minutes before swabbing.



Present the QR code on the bag to the technician.



Scan the QR code presented by technician using your YOTI, in order to link the test with your digital identity.



Open your FRANKD bag.



Take out the sterile swab.



Take the throat sample with the swab.



Open the transparent tube.



Put the swab inside the plastic tube and stir.



Close the tube and dispose of your swab in the designated (biohazard) bin.



Place the sample tube in the bag and close that bag.



Get your results in 25-30 minutes straight to your YOTI.